

# ENZYMEX L7

NEW VERSION

NEUTRAL ENZYMATIC DETERGENT

SURGICAL INSTRUMENTS



WASHING MACHINE



## ■ PRODUCT BENEFITS

- Neutral pH in operating conditions which allows cleaning all types of materials
- Stable tri-enzymatic complex combined with powerful surfactant
- The efficacy is guaranteed even in hard water
- CMR\*, NTA\*\* free
- Can be used in ultrasounds vats

\* Carcinogen, Mutagen  
substance toxic for Reproduction

\*\* Nitro-acetic Acid



**Franklab®**  
notre expertise l'Ultra-Propreté



www.franklab.com

# ENZYMEX L7

## NEUTRAL ENZYMATIC DETERGENT

### INSTRUCTION OF USE

#### Disinfecting washer, cabin and washing tunnel use:

- Recommended concentration : 0.2 to 0.8%
- Recommended temperature : ..40°C to 60°C
- Recommended time contact : ...3 to 10 minutes

Concentrations rate can be adapted according to the installation type (washing machine, high-pressure cleaning...) and according to the soil.



#### Ultrasounds vats use :

- Recommended concentration : 0.2 to 0.8%
- Recommended temperature : ..40°C to 60°C
- Recommended time contact : ...10 to 15 minutes



Note : Do not exceed a temperature of 65°C to not inhibit enzyme action.

### PRECAUTIONS OF USE

See (FS 010244)

### FEATURES

- Form : **Liquid**
- Colour : **Blue**
- pH at 0.2% : **7,4**
- pH at 0.8% : **7**
- EXpiration date : **2 ans**

### COMPOSITION

- Non-ionic surfactants
- Enzymes ( Protease, Amylase and Lipase)
- Sequestrants
- Coloring

### PACKAGING

- ① Box of 3 low cans x 5 L .....réf. **01024415**
- ② Box of 4 X 5L .....réf. **01024420**
- ③ Box of 2 x10 L .....réf. **01024422**
- Can of 20 L .....réf. **01024421**
- Can of 200 L .....réf. **01024489**



Hazardous product - Comply with the precautions for employment. Always read the label and product information before use.