

# ENZYMEX L9

ENZYMATIC DISINFECTANT DETERGENT FOR THE  
PRE-DISINFECTION OF MEDICAL DEVICES



## ■ APPLICATION

- Operating room, endoscopy, ophthalmology and contact lens

## ■ PRODUCT BENEFITS

- Tri-enzymatic complex (lipase, amylase and protease) combined with effective surfactants
- Instrumentation integrity guaranteed for 72 hours in immersion-bath
- Can be used in ultrasonic vat
- Activity tuberculocidal
- Proven efficiency on biofilm



# ENZYMEX L9

## ENZYMATIC DISINFECTANT DETERGENT FOR THE PRE-DISINFECTION OF MEDICAL DEVICES

### ■ INSTRUCTIONS FOR USE

#### ■ Preparation of the pre-cleaning bath:

- Wear suitable personal protective equipment ( see Safety Data sheet)
- Dilute the appropriate volume of **ENZYMEX L9** in 5L of water and homogenize the solution (see dilution table)

dose for 5L	Solution concentration		
	0.5%	2%	5%
	25 ml	100 ml	250 ml

#### ■ Use:

- 1- Soiled instruments have to be fully immersed
- 2- Scrub medical devices and use a swab if necessary
- 3- Contact time: according to the antimicrobial activity desired (see disinfecting properties)
- 4- Rinse thoroughly

*Renew the bath for each use*

### ■ DISINFECTING PROPERTIES

Activity	Norms	Concentration	Contact time
<b>Bactericidal</b>	EN 13727 +A1	0.5%	5 minutes
	EN 14561	0.5%	10 minutes
<b>Yeasticidal</b>	EN 13624	0.5%	5 minutes
	EN 14562	0.5%	10 minutes
<b>Virucidal</b>	EN 14476 + A2 ( <i>Herpes Virus</i> )	0.5%	5 minutes
	EN 14476 + A1 ( <i>DBV Virus</i> <i>Hepatitis C model, HIV Virus</i> )	0.5%	10 minutes
	EN 17111 ( <i>Vaccinia Virus</i> )	0.5%	10 minutes
<b>Tuberculocidal</b>	EN 14348	2%	60 minutes
		5%	10 minutes
	EN 14563	2%	60 minutes
		5%	10 minutes

### ■ PRECAUTIONS OF USE

See Safety Data Sheet

### ■ FEATURES

- Form : **Liquid**
- Color: **Green**
- pH at 0.5%: **7.5**
- Expiration date: **2 years**

### ■ COMPOSITION

- Non-ionic surfactants
- Quaternary ammonium
- Enzymes: lipase, amylase and protease
- Sequestrants

### ■ PACKAGING

- 1 Box of 250 dosis of 25ml.....ref. **1043547**
- 2 Box of 12 x1L.....ref. **1043516**
- 3 Box of 4 x 5L.....ref. **1043520**



Hazardous product - Follow precautions of use. Read label and product information before use. Medical device Class IIb.