

AEROSEPTINE 4

Aerosol

"ONE SHOT" CAN OF READY-TO-USE DISINFECTANT FOR AERIAL TREATMENT OF SURFACES AND FURNITURE

AIRBORNE DISINFECTION



■ FIELDS OF APPLICATION

■ Health establishments, animal care houses, laboratories

■ PRODUCT ADVANTAGES

- Easy to use
- Alternative to heavy fogging equipment
- Minty smell
- No oxidation of the surfaces



AEROSEPTINE 4 *Aerosol*

"ONE SHOT" CAN OF READY-TO-USE DISINFECTANT FOR AERIAL TREATMENT OF SURFACES AND FURNITURE

■ INSTRUCTIONS FOR USE

■ Preparation :

- Concentration : 50 ml / 20 m³
- Recovery time : 2 hours

■ Use :

- 1- Remove the cap, turn the can upside down and tap the white tip on a hard surface.
- 2- Place the can on an elevated support, putting it back in its normal position, head up.
- 3- Let the aerosol diffuse and leave the room.
- 4- Diffusion is automatic until the aerosol is completely emptied.

■ DISINFECTING PROPERTIES

Activity	Standards	Diffusion time
Bactericidal	NFT T 72-281	45 seconds (until the aerosol is completely emptied)
Yeasticidal	NFT T 72-281	45 seconds (until the aerosol is completely emptied)

■ PRECAUTIONS OF USE

See (SDS 010458)

■ FEATURES

- Aspect : Liquid
- Color : Incolore
- Odor: mint

■ COMPOSITION

- Quaternary ammonium
- Mint fragrance
- Propellant

■ PACKAGING

- 1 Box of 32 One Shot cans of 50 mL..... ref. **10458114**



Hazardous product - Follow precautions of use. Use biocides cautiously. Always read label and product information before use.