

# CA 5 P

ACID DESCALING DETERGENT FOR  
BEDPAN WASHER DISINFECTORS

BEDPAN WASHERS



## ■ PRODUCT BENEFITS

- Powerful detergent and descaling action
- Prevention of scale deposits in the steam generator, spindles, the water tank and the tub.
- Compatible with glass, plastic materials and steel...
- Decontamination of the radioactivity



## ACID DESCALING DETERGENT FOR BEDPAN WASHER DISINFECTORS

### ■ INSTRUCTIONS FOR USE

- Recommended concentration: 0.1 to 0.5% depending on water hardness
- Recommended temperature: 45 to 95 °C

#### ■ Use:

- 1- Unscrew the cap of the container and plunge the sampling probe in the liquid.
- 2- Put the 5L container in the plinth of the washer.
- 3- Start the cycle.

*Monitor the liquid volume of the container. Consumption, depending on water hardness, must be 1 to 5mL/L.*  
*CA 5 P may also, if necessary, be used by soaking (concentration of use between 1 and 3%)*

### ■ PRECAUTIONS OF USE

See Safety Data Sheet (SDS 010525)

### ■ FEATURES

- Aspect: **Liquid**
- Color: **Translucent**
- Pure pH: **1.0**
- pH at 0.2%: **2.9**
- Expiration date: **4 years**

### ■ COMPOSITION

- Sequestrants of calcium and mineral salts

### ■ PACKAGING

- 1 Box of 4 x 5L.....ref. **1052520**
- 2 box of 3 low x 5L.....ref. **1052515**



*Hazardous product - Follow precautions of use. Always read label and product information before use. Medical device Classe 1.*