

PERALEX 9 Hecto +

PERACETIC ACID-BASED READY-FOR-USE DISINFECTANT



■ APPLICATION

- Operating room, endoscopy, optical equipment

■ PRODUCT BENEFITS

- Ready-for-use (activator free)
- High-level Disinfection in 10 minutes
- Compatible with materials composing endoscope disinfection benches, especially Corian.
- Aldehyde free formula: no fixation of proteins

Fb Franklab®
notre expertise l'Ultra-Propreté



READY
FOR USE

CE
0459



www.franklab.com

PERALEX 9 Hecto +

PERACETIC ACID-BASED READY-FOR-USE DISINFECTANT

INSTRUCTIONS FOR USE

Preparation of the disinfection bath:

- Wear reusable gauntlet gloves, goggles, apron and face protection
- Pour the required quantity of the ready-for-use solution in a vat in such a way that the material to disinfect is fully immersed

Use:

- 1- Medical device has to be immersed
- 2- The disinfectant must circulate inside the endoscope channels
- 3- Contact time: 10 minutes
- 4- Rinse thoroughly with water bacteriologically controlled
Renew the bath at most every 7 days. Check the stability of the bath at least every morning before first use with a PERATEST 500 strip. (TDS 760_P9H+).

DISINFECTING PROPERTIES

Activity	Norms	Concentration	Contact time
Bactericidal	EN 13727 /	100%	1 minute
	EN 14561	100%	5 minutes
Fungicidal	EN 13624 / EN 14562	100%	5 minutes
Virucidal	EN 14476	100%	10 minutes
Mycobactericidal	EN 14348 / EN 14563	100%	10 minutes
Sporicidal	EN 14347	100%	10 minutes
	EN 17126	100%	10 minutes

PRECAUTIONS OF USE

See (SDS 010474)

FEATURES

- Aspect: Liquid
- Color: Colorless
- Pur pH: 2.5
- Expiration date: 1 year

COMPOSITION

- Peracetic acid
- Hydrogen peroxide
- Acetic Acid

PACKAGING

- ① box of 4 x 5Lref. **1047420**



Hazardous product - Follow precautions of use. Always read label and product information before use. Medical device Class IIb.