

# DDN Surf

NEUTRAL DISINFECTANT DETERGENT FOR THE  
BIOCLEANING OF FLOORS AND SURFACES

SURFACES TREATMENT



FLOORS AND SOAKING



## PRODUCT ADVANTAGES

- Neutral pH disinfectant detergent
- Active on *Clostridium Difficile*
- Compatible with a wide range of materials
- Mint fragrance
- Biocide registration: TP 02 (inventory N°: 32082)
- Active on enveloped viruses

**Fb Franklab®**  
notre expertise l'Ultra-Propreté



[www.sterifrance.com](http://www.sterifrance.com)

# DDN Surf

## NEUTRAL DISINFECTANT DETERGENT FOR THE BIOCLEANING OF FLOORS AND SURFACES

### INSTRUCTIONS FOR USE

#### Preparation of the biocleaning solution:

- Wear suitable personal protective equipment.
- For the biocleaning of floors and surfaces in the medical sector:**  
Dilute 20ml of DDN Surf in 8L (0.25%) of water and homogenize the solution
- For the biocleaning of floors and surfaces in the food, industrial sectors and community:**  
Dilute 80ml of DDN Surf in 8L (1%) of water and homogenize the solution  
(160ml of DDN Surf in 8L of water (2%) for a fungicidal activity)
- For the decontamination of radioactivity:**  
Dilute 500ml of DDN Surf in 5L (10%) of water and homogenize the solution

#### Use :

- Clean the floor or surfaces or medical devices with a broom or a wipe or use the product by soaking
- Contact time: according to the antimicrobial activity desired (see disinfecting properties).
- Rinse thoroughly with water when using the DDN Surf by soaking.  
For the other applications, rinsing is not required *unless specified by the flooring manufacturer's instructions.*

### DISINFECTING PROPERTIES

Activity	Standards	Concentration	Contact time
Bactericidal	EN 13727 + A1	0.25%	5 minutes
	EN 1276 / EN 13697	1%	5 minutes
Yeasticidal	EN 13624	0.25%	5 minutes
	EN 1650	0.5%	5 minutes
	EN 13697	1%	5 minutes
Fungicidal	EN 13697	2%	10 minutes
	EN 1650	2%	15 minutes
Virucidal	EN 14476 PRV (hepatitis B model virus)	0.25%	1 minute
	EN 14476 + A1 HIV	0.25%	15 minutes
Sporicidal	EN 13704 C.difficile	1%	15 minutes

### PRECAUTIONS OF USE

See Safety Data Sheet

### FEATURES

- Aspect: **Liquid**
- pH at 0.25%: **6.4**
- pH at 10%: **7.0**
- Color: **Green**
- pH at 2%: **7.0**
- Expiration date: **2 years**

### COMPOSITION

- Non-ionic surfactants
- Sequestrants
- Quaternary ammonium
- Mint essential oil

### PACKAGING

- Box of 12 x 1L .....ref. **1076016**
- Box of 4 x 5L.....ref. **1076020**



Hazardous product - Follow precautions of use. Use biocides cautiously. Always read label and product information before use.