

# FB Wipes

WIPES FOR THE BIOCLEANING OF SURFACES

SURFACES TREATMENT



WIPES



## ■ PRODUCT ADVANTAGES

- Neutral pH impregnating solution
- Suitable for external structures of hemodialysis generators
- CMR\* free, colorant free, fragrance free
- Ready-to-use
- Cleaning and disinfection of surfaces coming into contact with foodstuffs
- Active on *Bacillus Subtilis* and on *Clostridium Difficile*
- Compatible with a wide range of materials
- Effective against enveloped virus

\* Cancerigen, Mutagen substance  
toxic for Reproduction

FOOD CONTACT



**Fb Franklab**  
notre expertise l'Ultra-Propreté



READY  
FOR USE

## WIPES FOR THE BIOCLEANING OF SURFACES

### INSTRUCTION FOR USE

#### First use of the wipes:

- Wear gloves and turn the bucket upside down before first use.
- Remove the lid, open the bag and get the first wipe out of the center of the roll et lead it through the opening of the lid.

#### Use :

- 1- Take one or several wipes out depending on the surface to be treated.
- 2- Rub the soiled surface, making sure the surface remains wet.  
Contact time: according to the antimicrobial activity desired (see disinfecting properties)
- 3- Let it dry or wipe it with a dry non-woven wipe.

*Rinsing is not required unless specified by the flooring manufacturer's instructions or for surfaces in contact with foodstuffs.*

### DISINFECTING PROPERTIES

Activity	Standards	Concentration	Contact time
<b>Bactericidal</b>	EN 1276/ EN 13727 + A2 / EN 13697	100%	1 minute
	EN 14561	100%	5 minutes
<b>Yeasticidal</b>	EN 13624 / EN 1650 +A1	100%	1 minute
	EN 13697 / EN 14562	100%	5 minutes
<b>Fungicidal</b>	EN 13697	100%	5 minutes
	EN 1650 +A1 / EN 13624 / EN 14562	100%	10 minutes
<b>Virucidal</b>	EN 14476 +A1 <i>Rotavirus, PRV (hepatitis B model virus), BVDV (hepatitis C model virus), VRS Virus</i> EN 14476 +A2 <i>Vaccinia Virus</i>	100%	5 minutes
<b>Sporicidal</b>	NFT 72-230 / EN 13704 <i>B.Subtilis and Clostridium Difficile</i>	100%	15 minutes

### PRECAUTIONS OF USE

See Safety Data Sheet

### FEATURES

- Aspect: **White** wipes
- pH of the impregnating solution: **6±1**
- Expiration date: **3 years**

### COMPOSITION

- Non-ionic surfactants
- Quaternary ammoniums

### PACKAGING

- 1 Box of 2 buckets of 200 wipes.....ref. **10785147**

1



Use biocides cautiously. Always read label and product information before use.