

# LUBRIX *Spray*

PARAFFIN OIL BASED LUBRICANT  
FOR SURGICAL INSTRUMENTS

SURGICAL INSTRUMENTS



## ■ PRODUCT BENEFITS

- Manual lubrication of the joints of surgical instruments
- Effective to prevent blackish spots resulting from a poor water quality
- Brilliance of treated surfaces
- Easy and precise maintenance of the instruments guaranteed



**Fb Franklab**<sup>®</sup>  
notre expertise l'Ultra-Propreté



READY  
FOR USE

[www.franklab.com](http://www.franklab.com)

# LUBRIX *Spray*

## PARAFFIN OIL BASED LUBRICANT FOR SURGICAL INSTRUMENTS

### ■ INTRUCIONS FOR USE

#### ■ First use:

- Wear protective gloves/ eye protection
- Put the actuator on top of the valve

#### ■ Instructions for use:

- 1- Agitate the aerosol before use
- 2- Hold the aerosol vertically and pulverize in a targeted way the joints of the instruments with short pressions
- 3- Manipulate the joints several times to ensure a good spread of the product
- 4- Dry wipe

### ■ PRECAUTIONS OF USE

See (SDS 010328)

### ■ FEATURES

- Aspect: **Liquid**      ■ Color: **Colorless**
- pH: **6.5**
- Conservation after opening: **2 years**

### ■ COMPOSITION

- Purified paraffin oil
- Emulsifier
- Propellant: nitrogen

### ■ PACKAGING

- ① Box of 4 aerosol of 400ml .....ref. **1032856**



Hazardous product - Follow precautions of use. Always read label and product information before use.