

FB Spray

DISINFECTANT DETERGENT FOR HIGH SURFACES

BIOCLEANING



SPRAY

FOOD CONTACT

+

PRODUCT BENEFITS

- Neutral pH solution
- CMR *, colorant, fragrance free
- Active on enveloped virus
- Suitable for biocleaning of surfaces in contact with foodstuffs.
- Active on *Bacillus Subtilis* and on *Clostridium Difficile*
- Easy-to-use
- Compatible with a wide range of materials
- Biocide registration: TP 02/03/04(inventory N°: 65065)

* *Cancerigen, Mutagen substance
toxic for Reproduction*



Fb Franklab®
notre expertise l'Ultra-Propreté



READY-
TO-USE

www.sterifrance.com

DISINFECTANT DETERGENT FOR HIGH SURFACES

INSTRUCTIONS FOR USE

1st use of spray:

- Wear gloves and unlock the spray

Use:

- Spray uniformly **FB Spray** directly on the surface or on a dry single use wipe
- Spread the solution with the wipe
- Surface must be wet during 1 minute minimum
- Leave on to dry or wipe with dry single use wipe.

Product does not require rinsing, unless required otherwise by the manufacturer of surfaces or when in contact with foodstuffs.

DISINFECTING PROPERTIES

Activity	Standards	Contact time
Bactericidal	EN 13727 + A2 / EN 13697 / EN 1276	1 minute
	EN 14561	5 minutes
Yeasticidal	EN 13624 / EN 1650 + A1	1 minute
	EN 13697 / EN 14562	5 minutes
Fungicidal	EN 13697	5 minutes
	EN 1650 / EN 13624 / EN14562	10 minutes
Virucidal	EN 14476 +A1 Rotavirus, PRV (hepatitis B virus model), DBV (hepatitis C virus model), VRS Virus, EN 14476 +A2 Vaccinia Virus	5 minutes
Sporicidal	NFT 72-230 / EN 13704 B.Subtilis et Clostridium Difficile	15 minutes

PRECAUTIONS OF USE

See Safety Data Sheet

FEATURES

- Aspect: **Flowable liquid**
- pH: **6±1**
- Expiration date: **3 years**

COMPOSITION

- Non-ionic surfactants
- Quats

PACKAGING

- Box of 12 sprays of 750 ml.....ref. **1078640**
- Box of 12 bottles of 500 ml.....ref. **10786160**
- Box of 4 cans of 5 L.....ref. **1078620**



Use biocides cautiously, follow precautions of use. Always read label and product information before use.