

ENDEX P.A.E

READY-TO-USE LIQUID DISINFECTANT
BASED ON GLUTARALDEHYDE



■ PRODUCT ADVANTAGES

- Wide antimicrobial spectrum
- Quick disinfectant activity
- Compatibility with a wide range of materials
- Stability of the solution : 30 days

Fb Franklab®
notre expertise l'Ultra-Propreté



READY-TO-USE LIQUID DISINFECTANT BASED ON GLUTARALDEHYDE

■ INSTRUCTION FOR USE

■ Preparation of the disinfection bath :

- Wear reusable long-sleeved gloves, goggles, apron and mask
- Pour the required quantity of the ready-to-use solution into a tray so that the equipment to be disinfected is fully immersed

■ Use :

- 1- Immerse the medical device to be treated
- 2- Circulate the disinfectant inside the channels for endoscopes
- 3- Contact time: 10 to 60 minutes depending on desired spectrum
- 4- Rinse thoroughly with microbiologically controlled water

■ PROPRIÉTÉS DÉSINFECTANTES

| Activity | Standard time | Concentration | Contact time |
|------------------|----------------------------|---------------|--------------|
| Bactericidal | EN 13727 | 100% | 5 minutes |
| Yeasticidal | EN 13624 | 100% | 5 minutes |
| Fungicidal | EN 13624 | 100% | 5 minutes |
| Virucidal | EN 14476 <i>Poliovirus</i> | 100% | 5 minutes |
| Tuberculocidal | EN 14348 | 100% | 10 minutes |
| | EN 14563 | 100% | 10 minutes |
| Mycobactericidal | EN 14348 | 100% | 15 minutes |
| | EN 14563 | 100% | 15 minutes |
| Sporicidal | EN 14347 | 100% | 60 minutes |

■ PRECAUTIONS OF USE

Refer to the Safety Data Sheet (FS 010463)

■ FEATURES

- Aspect : Liquide
- pH : 6,3
- Expiration date : 2 ans
- Conservation after opening : 1 mois

■ COMPOSITION

- Glutaraldehyde (2%)

■ PACKAGING

- ① Box of 4 x 5 L cansref. **1046320**



Hazardous product - follow the precautions of use. Always read the label and product information before use.