


ECOSEPTOL FOAM

DISINFECTANT DETERGENT FOAM
ALCOHOL-FREE READY TO USE

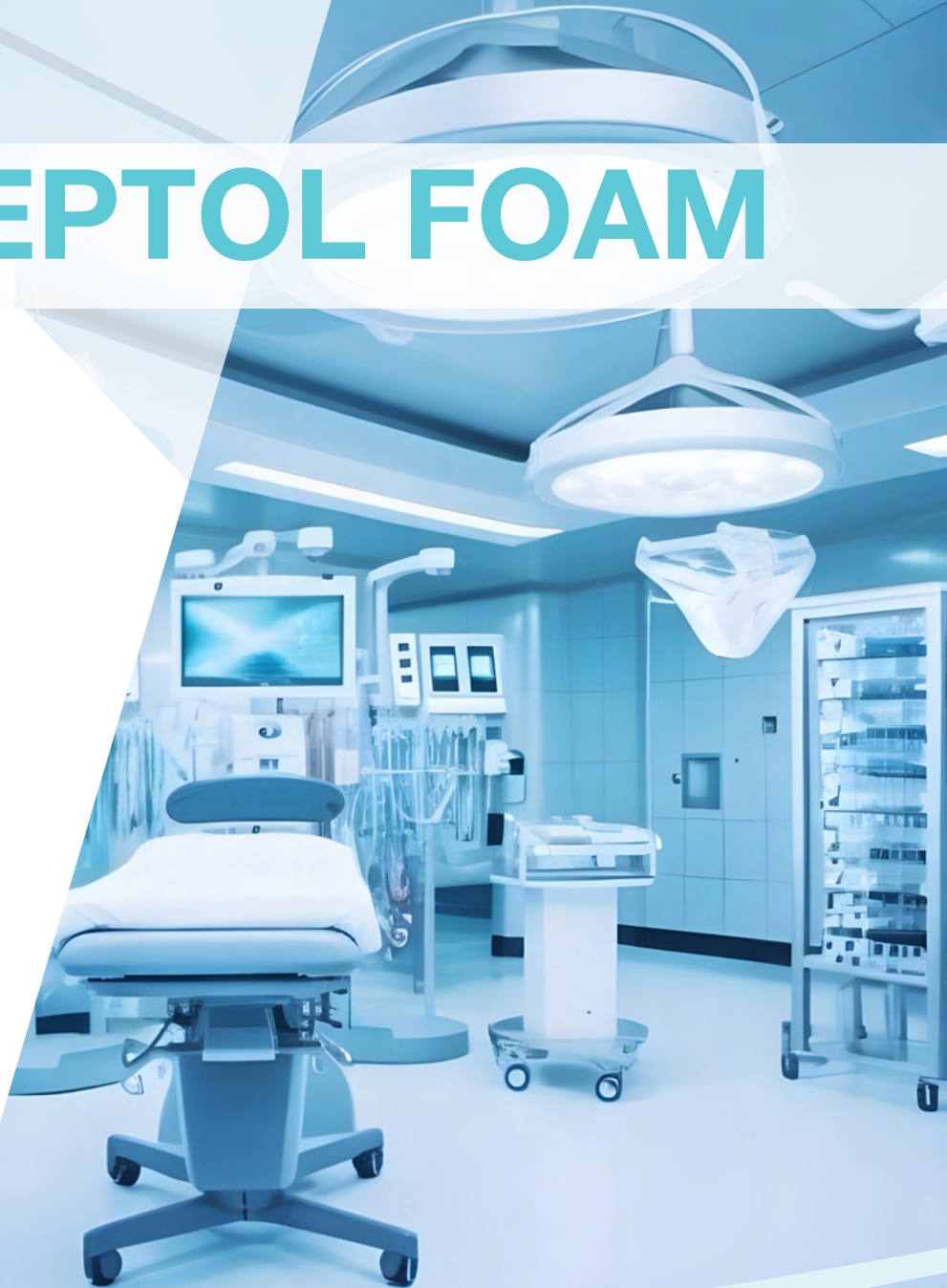

■ PRODUCT ADVANTAGES

- pH-neutral solution
- Phosphate-free, EDTA-free, dye-free, fragrance-free
- Compatibility with a wide range of materials thanks to its chloride-free formulation
- Effective against enveloped viruses and Clostridium Difficile



ECOSEPTOL FOAM

DISINFECTANT DETERGENT FOAM ALCOHOL-FREE READY TO USE

INSTRUCTIONS FOR USE

■ On first use :

- Wear suitable PPE and unlock the spray

■ Use :

- 1- Spray or evenly distribute Ecoseptol Foam directly on the surface to be treated or on a dry non-woven single-use wipe
- 2- Spread the solution/foam over the surface with the wipe

3- Contact time: Depending on the antimicrobial activity required

4- Do not rinse*

*Does not require rinsing, unless otherwise specified by the manufacturer of the surface coating treated or for surfaces in contact with foodstuffs.

5- Throw the support

6- Repeat whenever necessary

MICROBIOLOGICAL PROPERTIES

Activity	Standards	Strains	Contact time
Bactericidal	EN 16615	<i>Staphylococcus aureus</i> , <i>Pseudomonas aeruginosa</i> , <i>Enterococcus hirae</i>	2 minutes
	EN 13727		5 minutes
	EN 14561		10 minutes
	EN 1276	<i>Staphylococcus aureus</i> , <i>Pseudomonas aeruginosa</i> , <i>Enterococcus hirae</i> , <i>E.coli</i>	2 minutes
	EN 13697		5 minutes
	EN 13727	<i>Enterococcus faecium</i> ERV	5 minutes
Yeasticidal	EN 16615	<i>Candida albicans</i>	2 minutes
	EN 13624		
	EN 1550		5 minutes
	EN 13697		10 minutes
	EN 14562		
	EN 13624	<i>Candida auris</i>	5 minutes
Fungicidal	EN 13624	<i>Aspergillus brasiliensis</i>	
	EN 13697		15 minutes
	EN 1650		
Virucidal against enveloped viruses ⁽¹⁾	EN 14476	<i>Vaccinia virus</i>	5 minutes
Limited-spectrum virucide ⁽²⁾		<i>Norovirus</i> , <i>Adenovirus</i>	15 minutes
Active on Rotavirus, PRV, BVDV		<i>Rotavirus</i> , <i>PRV virus (model hepatitis B virus)</i> , <i>BVDV virus (model hepatitis C virus)</i>	5 minutes
Sporicidal against <i>Clostridium difficile</i>	EN 13704	<i>Clostridium difficile</i>	
	EN 17126		15 minutes

(1): The test for virucidal activity against enveloped viruses (Vaccine Viruses) is representative of the enveloped viruses presented in Annex A of standards EN 14476+A2 and EN 16777 and in Annex B of standard EN 17111, such as: Hepatitis B virus (HBV); Hepatitis C virus (HCV); Human immunodeficiency virus (HIV); Herpesviridae (Herpes virus); Coronavirus; Influenza, rabies, rubella, measles viruses; ...

(2): The limited-spectrum virucidal activity test is representative of enveloped viruses (Annex A of standards EN 14476+A2 and EN 16777 and Annex B of standard EN 17111), and noroviruses, rotaviruses and adenoviruses.

PACKAGING

① Box of 12 sprays of 750 mL ref. **23030**

Product for professional use - follow the precautions for use. Before use, read the label and product information. Class IIb medical device

PRECAUTIONS FOR USE

Refer to the Safety Data Sheet

FEATURES

- Aspect : Flowable liquid
- pH : 6

COMPOSITION

- Cationic surfactants
- Non-ionic surfactants
- Chlorhexidine
- Sequestering agents
- Excipients

