

# ENZYMEX L7

TRI-ENZYMATIC DETERGENT FOR AUTOMATED  
PROCESS



## ■ PRODUCT BENEFITS

- Compatible with a wide range of materials
- Efficiency in hard water
- Can be used in ultrasonic cleaners
- PH-neutral tri-enzymatic complex
- Enzymex L7 contains at least 99.99% biodegradable materials



# ENZYMEX L7

## TRI-ENZYMATIC DETERGENT FOR AUTOMATED PROCESS

### INSTRUCTIONS OF USE

#### ■ Washer disinfectors, cabin and washing tunnel use:

- Recommended concentration :. From 0.2 %
- Recommended temperature : ....40°C to 60°C
- Recommended contact time : ....From 3 minutes

Parameters are adapted according to water quality and cleaning system.

#### ■ Ultrasounds vats use :

- Recommended concentration :. From 0.2 %
- Recommended contact time : ....From 3 to 5 minutes

### PRECAUTIONS OF USE

Refer to Safety Data Sheet

### COMPOSITION

- Non-ionic surfactants
- Enzymes ( Protease, Amylase and Lipase)
- Sequestrants
- Colorant

### FEATURES

- Form: **Liquid**
- Colour: **Blue**
- Pur pH : **6.90**
- pH at 0.2%: **7,40**
- pH at 1%: **7.50**

### PACKAGING

Box of 3 low cans x 5 L .....ref. **01024415**

① Box of 4 X 5L .....ref. **01024420**

② Box of 2 x10 L .....ref. **01024422**

Can of 20 L .....ref. **01024421**

Can of 200 L .....ref. **01024489**



Product for professional use - follow the precautions for use. Before use, read the label and product information. Class I Medical Device