

FB WIPES

CLEANING AND DISINFECTING WIPES



■ PRODUCT BENEFITS

- Treatment of surfaces and medical devices destined to control microbiological risks linked to the environment (e.g. bed, operating table, monitoring, hemodialysis generators, respirators, dental units)
- Adapted to large surfaces (FB wipes XXL)
- pH-neutral, fragrance-free impregnation solution
- Effective against enveloped viruses and active on *Bacillus subtilis* and *Clostridium difficile* spores
- Compatible with a wide range of materials including synthetic coatings

FB Wipes

CLEANING AND DISINFECTING WIPES

INSTRUCTIONS FOR USE

■ First use of the wipes:

- Wear gloves and remove the protective lid.
- Take out the 1st wipe with one hand **while holding the pack with the other hand.**

■ Use :

- 1- Clean the area by taking as many wipes as necessary
- 2- Ensure that the contact time is consistent with the desired antimicrobial activity.
- 3- Close the pack tightly after use. Do not reuse the wipe.

MICROBIOLOGICAL PROPERTIES

Activity	Standards	Strains	Contact time
Bactericidal	EN 16615	<i>Staphylococcus aureus, Pseudomonas aeruginosa, Enterococcus hirae</i>	2 min.
	EN 13727		1 min.
	EN 14561		5 min.
	EN 1276	<i>Staphylococcus aureus, Pseudomonas aeruginosa, Enterococcus hirae, E.coli</i>	1 min.
	EN 13697		1 min.
	EN 13727	<i>Enterococcus faecium VRE</i>	1 min.
Yeasticidal	EN 16615	<i>Candida albicans</i>	2 min.
	EN 13624		1 min.
	EN 14562		5 min.
	EN 13697		5 min.
	EN 1650		1 min.
	EN 13624	<i>Candida auris</i>	1 min.
Fungicidal	EN 13697	<i>Aspergillus brasiliensis</i>	5 min.
	EN 13624		10 min.
	EN 14562		
	EN 1650		
Virucidal against enveloped viruses ⁽¹⁾	EN 14476	<i>Vaccinia virus</i>	5 min.
Active on Rotavirus, PRV, BVDV, VRS		<i>Rotavirus, Virus PRV (hepatitis B virus model), Virus BVDV (hepatitis C virus model), Virus VRS</i>	
Sporicidal against Clostridium difficile	EN 17126 EN 13704	<i>Clostridium difficile</i>	15 min.
Active on Bacillus subtilis	EN 13704	<i>Bacillus subtilis</i>	

1 : The test for virucidal activity against enveloped viruses (Vaccine Viruses) is representative of the enveloped viruses presented in Annex A of standards EN 14476+A2 and EN 16777 and in Annex B of standard EN 17111, such as: Hepatitis B virus (HBV); Hepatitis C virus (HCV); Human immunodeficiency virus (HIV); Herpesviridae (Herpes virus); Coronavirus; Influenza, rabies, rubella, measles viruses; ..

PACKAGING

FB Wipes

① Box of 12 packs of 100 wipesref. **10785142**

FB Wipes XXL

② Box of 12 packs of 60 wipes.....ref. **10785145**

1



2



Observe safety precautions. Use biocides with care. Read the label and product information before use.